


Hand Magnifier instructions

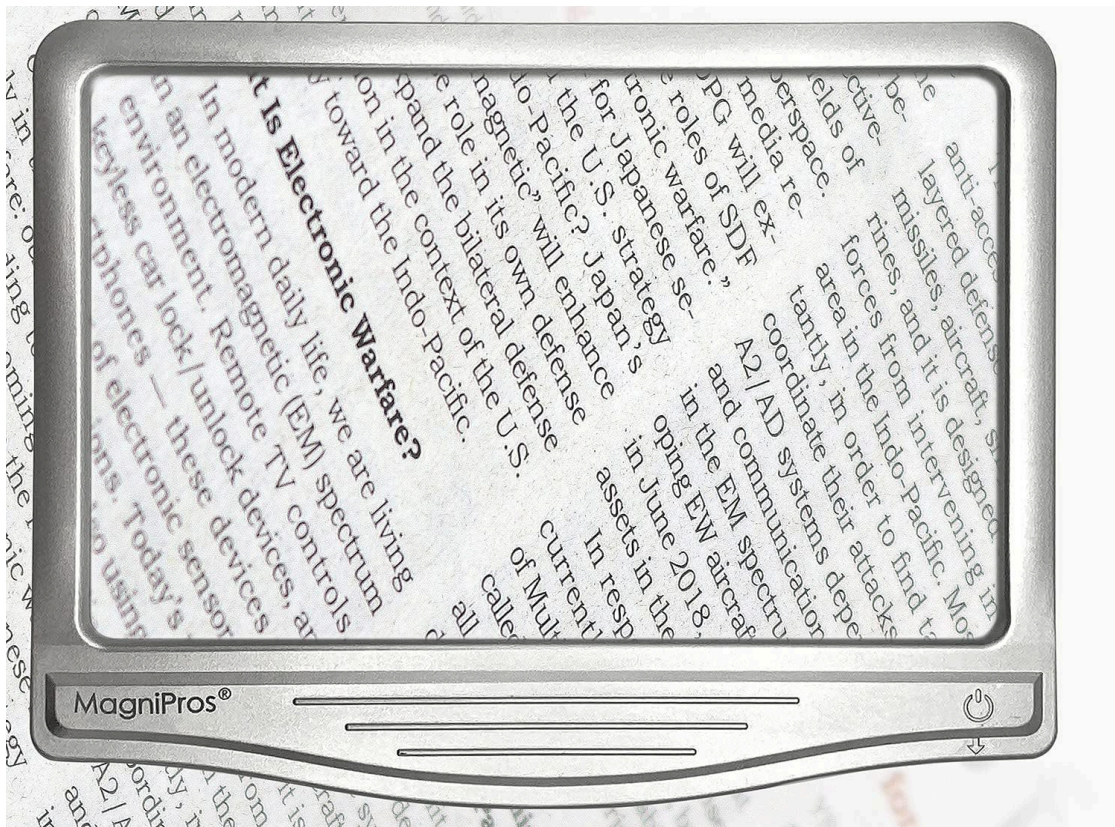

Instructions and basic use

Install batteries

Gently pull out the battery slot beneath the middle of the handle side of the magnifier. Load one battery on the left side and push it slightly to the left. Load another battery on the right side and push it slightly to the right. Align

the batteries with the flat side on the left. Load the center battery in the same orientation.

Turn light on

Turn the light on by sliding the dial to the right.

Spin the dial below the power button up or down to adjust the brightness.

Using the hand magnifier

Hold 2-4" away from the page/content you are viewing to read it clearly.

You may need to move the magnifier lens for it to focus properly.

Turn light off

Turn off by sliding the dial to the left until just after it clicks.

Manufacturer instructions

ANTI-GLARE LED 3X PAGE MAGNIFIER

BENEFITS OF OUR "SMD LED" LIGHTS:

Higher Brightness Level:

30% Brighter than traditional LEDs.

More Energy Efficient:

Consumes 60% less power than traditional lighting, therefore, a new set of batteries would last longer.

Longer Life Span:

Each SMD LED can last at least 50,000+ hours, 3 times than the traditional ones.

Provides More Evenly Lit Viewing Area:

SMD LED lights do not focus on one spot, lighting is scattered within certain area, which provides more evenly lit area and would be more comfortable to view objects from your naked eyes.

Step 1 Please peel off the protective film cover on the lens before using this led page magnifier to view objects.

There is only one protective film on the flat side of the lens.

Step 2



Follow polarity indicators(on battery cover) and insert 3 AAA batteries (not included) into battery compartment.


***Tip: Install batteries on both sides, then install the third battery in the center of the battery compartment.**

Step 3



To turn on or increase brightness level of led lights please turn the dimmable switch to the right, to turn off or decrease brightness level, please turn dimmable switch to the left.

Step 4

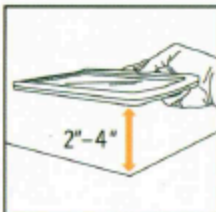

Please hold the led page magnifier 2" to 4 " above objects intended to magnify for optimal magnifying result.

Note: the film has already been removed.