



Page Magnifier instructions



Instructions and basic use

Take the unit out of its case.

Make sure it is loaded with three (3) AAA batteries. Turn the magnifier over. Gently pull the plastic tab in the upper right corner to reveal the battery tray. Load three (3) AAA batteries, all with their flat ends down into the tray. Slide each battery downward to make room for the others. The last battery will be a little tight. Replace the plastic battery tray cover.

Then,

Either: Extend the two halves of the legs to stand it up. Place it on a flat surface where you can comfortably look down at the magnification lens.

Or: Hold the unit at about chest level using the included/attached strap.

Place your reading material beneath the lens with a gap of about 2-3 inches between the text and the lens.

Looking down at the MagniPros magnifier, the lighting controls are on the side on the lower left corner. It will be in the middle OFF position.

Slide the brightness controller right for “gentle” 50% brightness.

Slide the brightness controller left for “bright” 100% brightness.

Slide the brightness controller to the middle to power the magnifier OFF.

The magnifier works via an unpowered lens, so if lighting conditions are adequate, you do not have to adjust the brightness.

Move content around underneath the lens to adjust the focus.

Remove your content from underneath the magnifier.

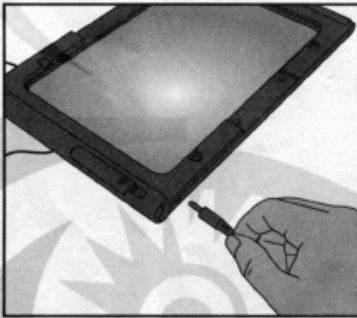
Slide the brightness controller to the middle to power the magnifier OFF.

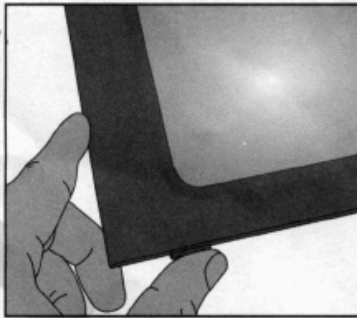
Manufacturer instructions

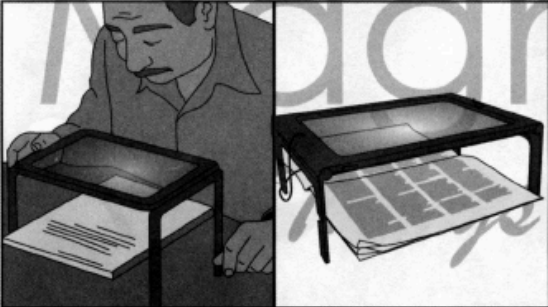
Instruction Manual
(Item#300PSL)

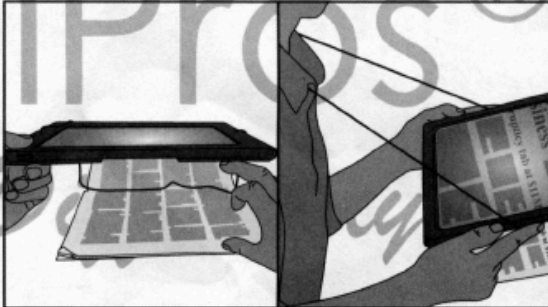
MagniPro 3. LED Hands Free Page Reading Magnifier

1. Decide how would you like to power this page magnifier. There are two ways to power this led page magnifier:
 - a. 3 AAA Batteries (NOT INCLUDED)
 - b. USB Power cord (INCLUDED)
2. 

If you were to power this page magnifier by 3 AAA batteries (NOT INCLUDED), please follow polarity indicators(in the battery compartment) and insert 3 AAA batteries and close the cover.
3. 

If you were to power this page magnifier by USB cord (included), please plug in the power cord at the bottom of this page magnifier(see figure 3 for your reference).
4. 

The power switch is located at the bottom left, where indicator display as “ on – off – on” , two brightness levels, turn to the right for 50% brightness level, turn to the left for 100% brightness, turn to the middle if you wish to turn off the lights.
5. 

Flip out the 4 legs to let this page magnifier to stand by itself, and put objects/small prints underneath it to magnify.
6. 

Hang this page magnifier on your neck with the included lanyard and adjust distance between this page magnifier and objects for your optimal magnifying result, typically from 2” to 4.5” away from objects.

Note: power cable not included in this kit. Please use 3 AAA batteries instead.